HR FUNKCIONALNE NAPOMENE - SIGURNOST

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GB Fitting and Maintenance Guide for Grip Rails by BISK S.A.

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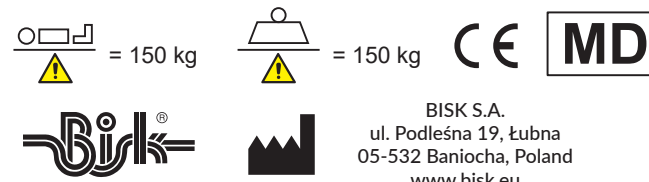
GB

Congratulations on your purchase of a durable and good quality product. (PRO handles; stools with backrest, stools, round stool, bath seat, shower chair) by BISK S.A. are a Class I medical product, meeting the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices. They bear the CE mark and a hygienic certificate of the National Institute of Hygiene. They comply with the provisions of the harmonised standard PN-EN 21856:2023.

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GB

The PRO series stools are a technical aid for people with limited mobility in daily personal hygiene. The person using the stool is able to sit comfortably on a higher plane without having to bend down deeply, sit in the bathtub or rise from a low position. There are no contraindications for the use of the product. The product is a technical aid for people with limited mobility for daily personal hygiene. They make it easier to perform various hygiene activities for people with limited movement functions. They improve safety by reducing the risk of falls. For this reason, we recommend their installation and use in all types of hygienic and sanitary facilities. The manufacturer guarantees stability and a static load-bearing capacity (acting vertically) of 1500 N (150 kg), provided that these products are installed correctly. The backrest stools have been tested under an increased load of 2250 N (225 kg). Installation according to the diagram below:

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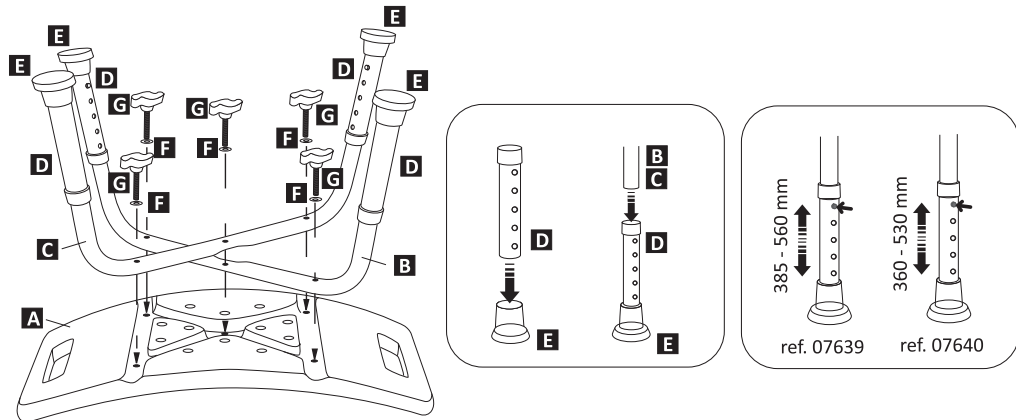
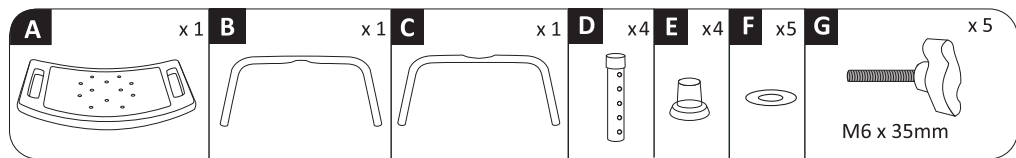
BISK S.A.
ul. Podleśna 19, Łubna
05-532 Baniocza, Poland
www.bisk.eu

GB It is recommended to entrust the installation to a suitably qualified person. Proper, stable and secure installation is a guarantee of the safe use of these products. The manufacturer shall not be held liable for malfunctions or accidents in the event of any alteration to the product. If, during use, the construction of the stools raises any doubts as to their durability or they have become unstable and could endanger the safety of persons using the stools - they must not be used. The products of the PRO series by BISK S.A. are manufactured and inspected according to strict quality and strength requirements. They are made of aluminium in combination with HDPE components. The stools are not intended to be used as gymnastic or any other purpose handle incompatible with their dedicated use. Any serious incidents with the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides.

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GB Hygiene rules: Clean with damp soft cloth, preferably soaked with washing-up liquid and wipe dry. Do not use chlorine-, acid- or nitro-solvent-based or abrasive agents. Only alcohol-based disinfectants may be used for disinfection.

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GB Maintenance rules: The circular stool should be periodically inspected for correctness of its construction (the manufacturer recommends maintenance in this respect once a year). Maintenance of stools' fastening should be performed by checking correctness of seating of stools' leg positioning pins and elimination of possible looseness of bolts connecting the seat with the stools' frame structure. The use of the product should be discontinued if any damage occurs (e.g. degradation of the material, breakage, damage to the attachment, damage to the user's support, mechanical wear and tear of the operating elements).

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GB Guarantee.

The Producer grants a 5-year guarantee. The guarantee does not cover any damages resulting from improper use or damages to the decorative coating caused mechanically or by the application of abrasive or chemically aggressive detergents during product maintenance.

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GB FUNCTIONAL REMARKS – SAFETY

In order to use the seats (chairs and stools) safely it is necessary to:

- Verify the structure of the seat (whether the product is complete, not damaged, whether the frame structure is not damaged).
- Verify possible damage to plastic elements (seat, backrest).
- Verify the correct height of the rack legs - use of the seat is allowed only when the legs are set at one height.
- Verify the correct installation and check there are no damage to the anti-slip pads (on all product legs).

To avoid injuries and possible accidents:

- Use the seats when they are properly screwed in accordance with the instructions and permanently attached to the frame.
- Verify the technical condition of the seats each time before use (checking the seat, backrest, rubber pads, verification of the structure and correct assembly, tightening of the screws).
- Use the products only on an even, hard, flat surface so as to prevent free movement of the seats when you sit down.
- Pay particular attention when using the seats on a wet/slippy surface, despite correctly fitted anti-slip feet.
- People with disabilities should be assisted to use the seat.

It is forbidden to:

- Rapidly jump on/off a chair/stool.
- Carry the chair/stool with the user.
- Stand with your feet on any element of the stool/chair, including the seat.
- The chair/stool's load limits are exceeded by the manufacturer.

Recommendation:

- To increase the safety of using the products, it is recommended to cover the slippy surfaces (padding pool, cabin) with non-slip mats.

In case of:

- any damage is found.
- lack of required components.
- instability of a correctly assembled product.

It is forbidden to use the product (please contact the seller). It is forbidden to repair the product yourself.

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