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### GB Hygiene rules:

Clean with damp soft cloth, preferably soaked with washing-up liquid and wipe dry. Do not use chlorine-, acid- or nitro-solvent-based or abrasive agents. Only alcohol-based disinfectants may be used for disinfection.

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### GB Maintenance rules:

The handles should be periodically inspected for correct fixing (the manufacturer recommends maintenance in this respect once a year). Maintenance of the fixing of the handles should be carried out by gently pushing back the handles, tightening the screws if there is any play and re-fixing the handles. The use of the product should be discontinued if any damage to the handles occurs (e.g. degradation of the material, breakage, damage to the attachment, damage to the user's support, mechanical wear and tear of the operating elements).

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GB Guarantee. The Producer grants a 5-year guarantee. The guarantee does not cover any damages resulting from improper use or damages to the decorative coating caused mechanically or by the application of abrasive or chemically aggressive detergents during product maintenance.

\* This does not apply to the middle\_xing element of the "L" bar (Ref. No. 04784 and 04785), where the mounting plate should be positioned according to Drawing 1C.

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BISK S.A. ul. Podleśna 19 Łubna, 05-532 Baniocza
## Instrukcja montażu, konserwacji i używania dla uchwytów firmy BISK S.A.
GB Fitting and Maintenance Guide for Grip Rails by BISK S.A.
DE Montage- und Wartungsanleitung für Gri\_e von BISK S.A.
HR Upute za montažu i održavanje rucka brenda BISK S.A.
SK Návod na montáž a údržbu madiel\_ rmy BISK S.A.