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GB**Guarantee.**

The Producer grants a 5-year guarantee. The guarantee does not cover any damages resulting from improper use or damages to the decorative coating caused mechanically or by the application of abrasive or chemically aggressive detergents during product maintenance.

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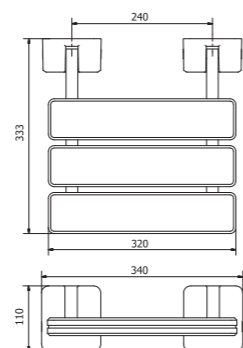
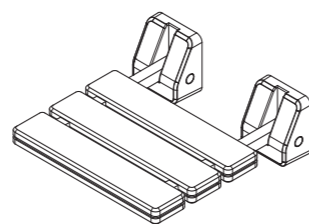
GB Fitting and Maintenance Guide for Grip Rails by BISK S.A.

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
	ref.	waga [kg] weight
Krzeselko prysznicowe DUO PRO Shower seat DUO PRO	07623	1,6

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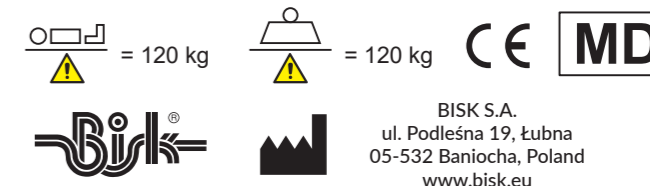GB Congratulations on your purchase of a durable and good quality product. Folding shower seat by BISK S.A. are a Class I medical product, meeting the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices. They bear the CE mark and a hygienic certificate of the National Institute of Hygiene. They comply with the provisions of the harmonised standard PN-EN 21856:2023.

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GB

The folding shower seat from the PRO series provides technical assistance to people with reduced mobility for daily personal hygiene. The person using the seat is able to sit comfortably on a higher plane without having to bend down deeply, sit in the bathtub or rise from a low position. There are no contraindications for the use of the product. The product is a technical aid for people with limited mobility for daily personal hygiene. They make it easier to perform various hygiene activities for people with limited movement functions. For this reason, we recommend their installation and use in all types of hygienic and sanitary facilities. The manufacturer guarantees stability and a static load-bearing capacity (acting vertically) of 1200 N (120 kg), provided that these products are installed correctly. The bathtub seat has been tested under an increased load of 1800 N (180 kg). Installation according to the diagram below:

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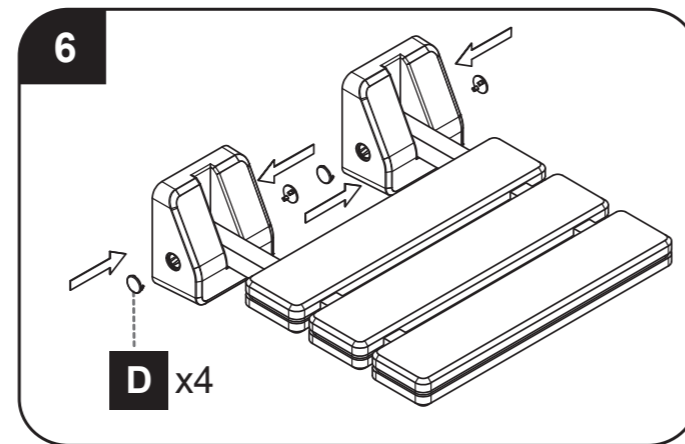
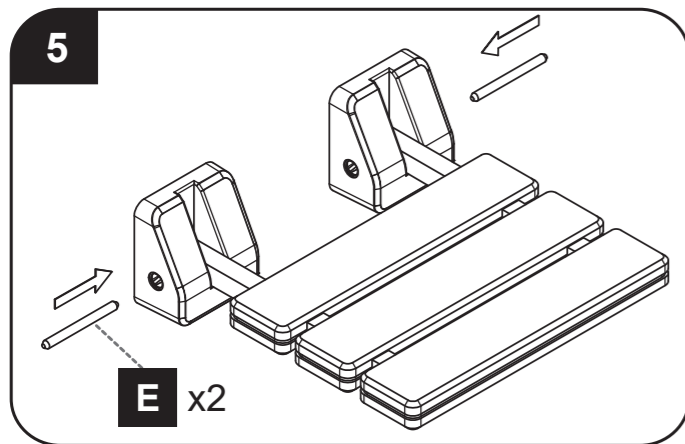
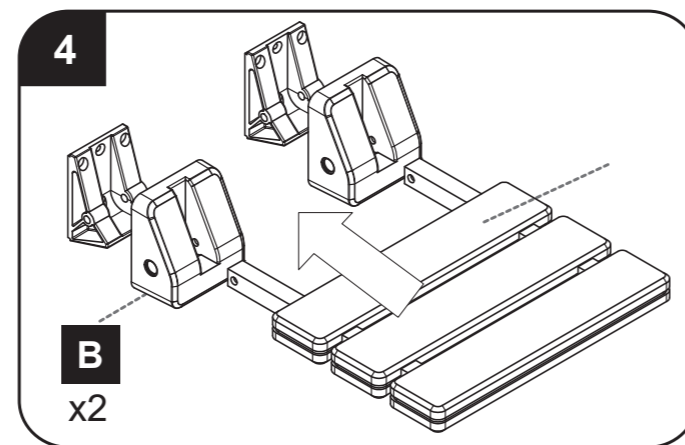
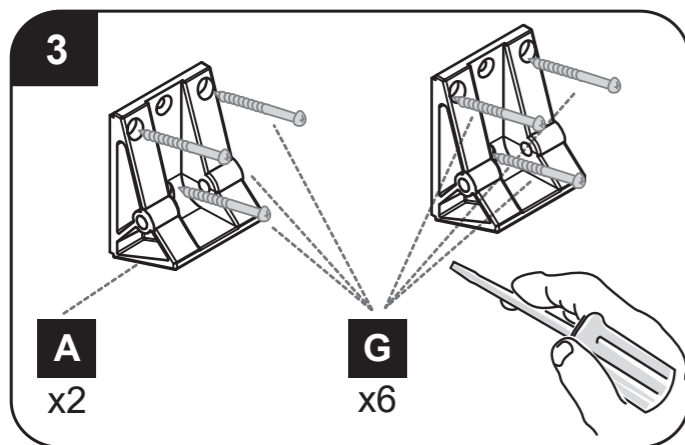
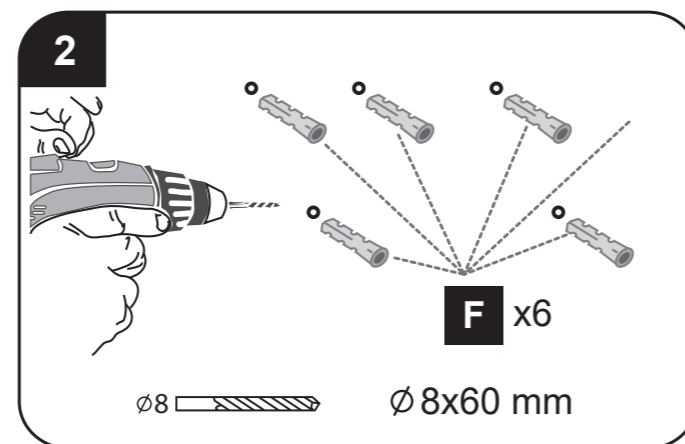
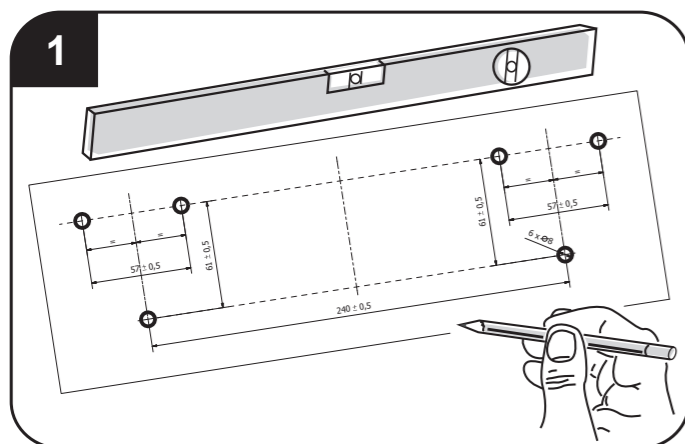
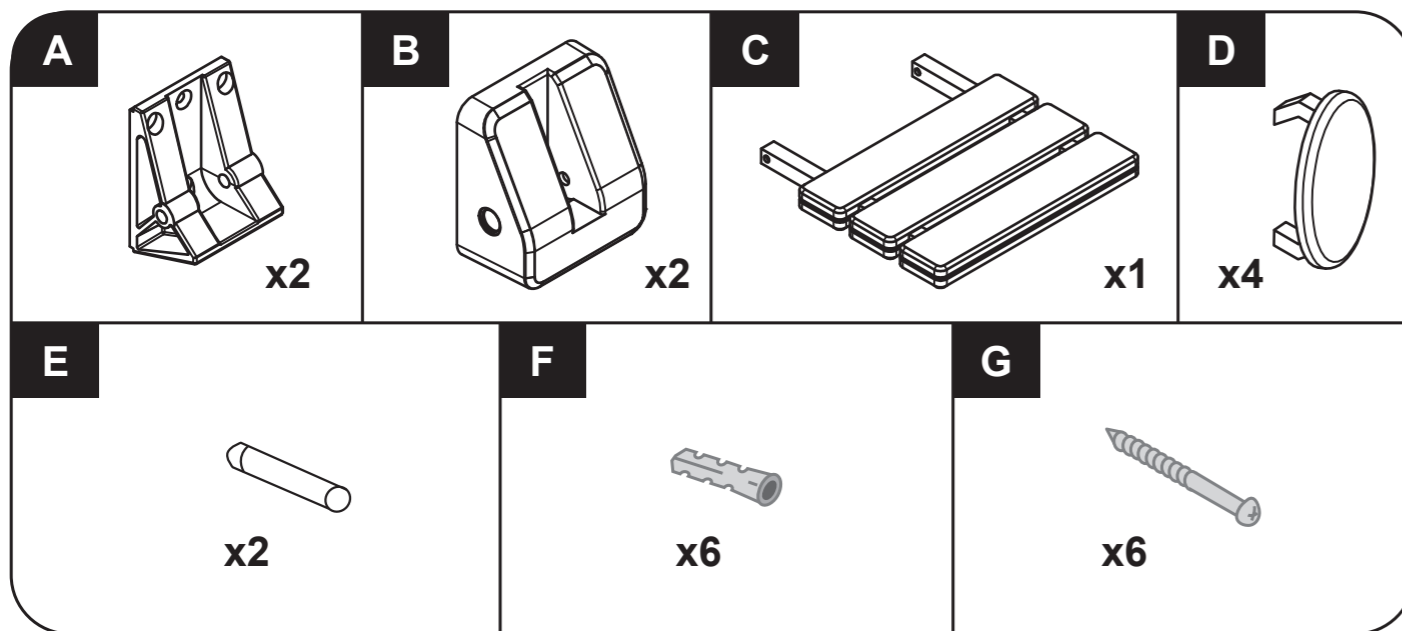
GB It is recommended to entrust the installation to a suitably qualified person. Proper, stable and secure installation is a guarantee of the safe use of these products. The manufacturer shall not be held liable for malfunctions or accidents in the event of any alteration to the product. If, during use, the fixing of the bathtub seat raises any doubts as to its strength or has become unstable and could endanger the safety of persons using the handle - it must not be used. The products of the PRO series by BISK S.A. are manufactured and inspected according to strict quality and strength requirements. It is made of aluminium in combination with HDPE elements. The bathtub seat is not intended to be used as gymnastic handle or any other purpose incompatible with its dedicated use. Any serious incidents with the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides.

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GB Hygiene rules:

Clean with damp soft cloth, preferably soaked with washing-up liquid and wipe dry. Do not use chlorine-, acid- or nitro-solvent-based or abrasive agents. Only alcohol-based disinfectants may be used for disinfection.

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GB Maintenance rules:

The folding shower seat should be periodically inspected for correctness of its construction (the manufacturer recommends maintenance in this respect once a year). Maintenance should be carried out by checking that the fixing screws are correctly seated under the covers. The use of the product should be discontinued if any damage of the bathtub seat occurs (e.g. degradation of the material, breakage, damage to the attachment, damage to the user's support, mechanical wear and tear of the operating elements).